

Blood Draw Instructions

Blood Draw Instructions - Multi-Fly Needle Method (New Kit)

1. Sterilize area
2. Insert multi-fly needle and stabilize
3. Fit adapter to other/open end
4. Remove foil cup to expose rubber stopper on 30 ml vial in preparation for draw.
5. Collect 2-3 ml of blood in small vacutainer tube and discard it.
6. Collect 20-25 ml of blood in the 30 ml vial
7. Complete label information
8. Place in plastic bag/biohazard bag
9. Store in refrigerator, DO NOT freeze, no more than 24 hours before shipping